

## Guarantee Card


FR 100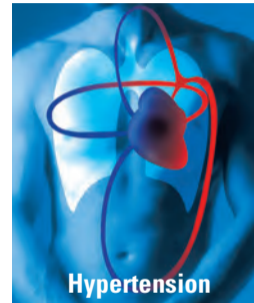


Hypertension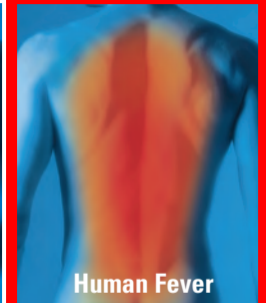

Human Fever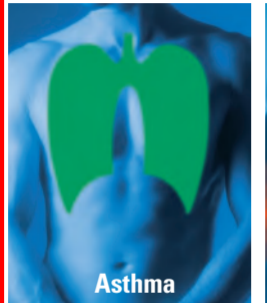

Asthma

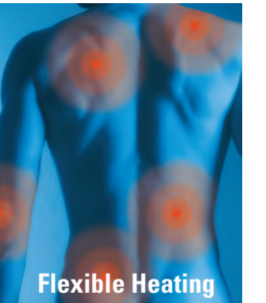Flexible Heating

## Europe / Middle-East / Africa

Microlife AG  
Esenstrasse 139  
9443 Widnau / Switzerland  
Tel. +41 / 71 727 70 30  
Fax +41 / 71 727 70 39  
Email admin@microlife.ch  
www.microlife.com

CE0044

VM 04

## Asia

Microlife Corporation.  
9F, 431, RuiGang Road, NeiHu  
Taipei, 11492, Taiwan, R.O.C.  
Tel. +886 2 8797-1288  
Fax +886 2 8797-1283  
Email service@microlife.com.tw  
www.microlife.com

## North / Central / South America

Microlife USA, Inc.  
1617 Gulf to Bay Blvd., 2nd Floor Ste A  
Clearwater, FL 33755 / USA  
Tel. +1 727 442 5353  
Fax +1 727 442 5377  
Email msa@microlifeusa.com  
www.microlife.com

IB FR 100 V16-5 1816

## Microlife FR 100







- EN → 2
- FR → 8
- ES → 14
- PT → 20
- DE → 26
- RU → 32
- UA → 38
- PL → 44
- HU → 50
- RO → 56
- NO → 62
- NL → 68
- FI → 74
- SV → 80
- DA → 86
- GR → 92

microlife®

## Microlife FR 100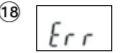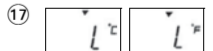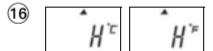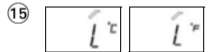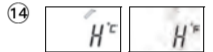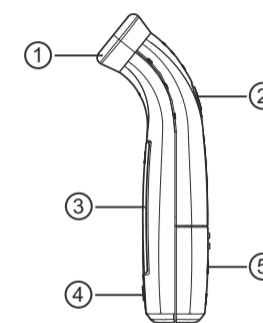

- ① Measuring sensor
- ② START button
- ③ Display
- ④ ON/OFF button
- ⑤ Battery compartment cover
- ⑥ All segments displayed
- ⑦ Memory
- ⑧ Ready for measuring
- ⑨ Measurement complete
- ⑩ Low battery indicator
- ⑪ Changing between Celsius and Fahrenheit

Read the instructions carefully before using this device.

Type BF applied part

Keep dry

- ⑫ Recall mode
- ⑬ Recall the last 12 Readings
- ⑭ Measured temperature too high
- ⑮ Measured temperature too low
- ⑯ Ambient temperature too high
- ⑰ Ambient temperature too low
- ⑱ Error function display
- ⑲ Blank display
- ⑳ Flat battery
- ㉑ Replacing the battery

The Microlife Forehead Thermometer is a high quality product incorporating the latest technology and tested in accordance with international standards. With its unique and clinically tested technology, the Microlife Forehead Thermometer can provide a stable reading comparable with an oral temperature. The device performs a self-test every time it is switched on to always guarantee the specified accuracy of measurements.

The Microlife Forehead Thermometer allows a simple and quick measurement of the body temperature. Please keep in mind that, – like any skin temperature –, the temperature at the forehead area might be lower than the core body temperature. Body temperature varies depending on the part of the body where it is measured. The variation can be between 0.2 - 1 °C.

Temperatures measured with different thermometers cannot be compared with one another. So, when taking your own temperature, advise your doctor which thermometer was used and in which part of the body.

A basic physiologic effect called vasoconstriction can occur in the early stages of fever, resulting in a cool skin effect,

Name of Purchaser / Nom de l'acheteur / Nombre del comprador / Nome do comprador / Name des Käufers / Ф.И.О. покупателя / П.И.П. покупателя / Imię i nazwisko nabywcy / Vásárló neve / Numele cumpărătorului / Kjøpers navn / Naam koper / Ostajan nimi / Inköparens namn / Forhandlers navn / Ονοματεπώνυμο αγοραστή

Serial Number / Numéro de série / Número de serie / Número de série / Serien-Nr. / Серийный номер / Серійний номер / Numer serijny / Sorozatszám / Număr de serie / Serijas numurs / Serienummer / Sarjanumero / Serienummer / Αριθμός σειράς

Date of Purchase / Date d'achat / Fecha de compra / Data da compra / Kaufdatum / Дата покупки / Дата покупки / Data zakupu / Vásárlás dátuma / Data cumpărării / Kjøpsdato / Datum van aankoop / Ostoräivämäärä / Inköpsdatum / Købsdato / Ημερομηνία αγοράς

Specialist Dealer / Revendeur / Vendedor especializado / Revendedor autorizado / Fachhändler / Специализированный дилер / Специализований дилер / Przedstawiciel / Forgalmazó / Distribuitor de specialitate / Specialist forhandler / Specialist Dealer / Alan kauppias / Återförsäljare / Specialforhandler / Εξειδικευμένος αντιπρόσωπος

the recorded temperature using a forehead thermometer can, therefore, be unusually low.

If the measurement result is not consistent with the patient's finding or unusually low, repeat the measurement every 15 minutes or double check the result by another core body temperature measurement, e.g. use an IR-ear drum thermometer or a rectal pen-type thermometer.

## Table of Contents

1. The Advantages of this Thermometer
2. Important Safety Instructions
3. How this Thermometer measures Temperature
4. Control Displays and Symbols
5. Directions for Use
6. Changing between Celsius and Fahrenheit
7. How to recall 12 readings in Memory Mode
8. Error Messages
9. Cleaning and Disinfecting
10. Battery Replacement
11. Guarantee
12. Technical Specifications
13. [www.microlife.com](http://www.microlife.com)  
Guarantee Card (see Back Cover)

## 1. The Advantages of this Thermometer

### Measures in a matter of seconds

The innovative infrared technology allows a temperature measurement simply by measuring the forehead. After 3 seconds the measurement will be displayed on the LCD.

### Accurate and reliable

The unique probe assembly construction incorporates an advanced infrared sensor, ensuring that each measurement is accurate and reliable.

### Gentle and easy to use

- The ergonomic design enables simple and easy use of the thermometer.
- This thermometer can even be used on a sleeping child without causing any interruption.
- This thermometer is quick, therefore child-friendly.

### Auto-Display Memory

The last reading is automatically displayed for 2 seconds when the unit is switched ON.

### Multiple readings recall

Users will be able to recall the last 12 readings when entering the recall mode, enabling efficient tracking of temperature variations.

### Safe and hygienic

- No risk of broken glass or mercury ingestion.
- Completely safe for use on children.
- Cleaning of the probe can be done with an alcohol-moistened cotton tissue, making this thermometer completely hygienic for use by the whole family.

### High Temperature Indication

10 short beeps and a red LCD backlight alert the patient that he/she may have a temperature equal to or higher than 37.5 °C.

## 2. Important Safety Instructions

---

- This device may only be used for the purposes described in these instructions. The manufacturer cannot be held liable for damage caused by incorrect application.
- **Never immerse this device in water or other liquids. For cleaning please follow the instructions in the «Cleaning and Disinfecting» section.**
- Do not use this device if you think it is damaged or notice anything unusual.
- Never open this device.
- This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the «Technical Specifications» section.
- Protect it from:
  - extreme temperatures
  - impact and dropping
  - contamination and dust
  - direct sunlight
  - heat and cold

- If the device is not going to be used for a prolonged period the battery should be removed.
- Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed. Be aware of the risk of strangulation in case this device is supplied with cables or tubes.
- The function of this device may be compromised when used close to strong electromagnetic fields such as mobile phones or radio installations and we recommend a distance of at least 1 m. In cases where you suspect this to be unavoidable, please verify if the device is working properly before use.

**WARNING:** Use of this device is not intended as a substitute for consultation with your physician. This device is NOT waterproof so do not immerse in liquids.

## 3. How this Thermometer measures Temperature

---

This thermometer measures infrared energy radiated from the forehead. This energy is collected through the lens and converted to a temperature value.

## 4. Control Displays and Symbols

---

- **All segments displayed** (6): Press the ON/OFF button (4) to turn on the unit; all segments will be shown for 2 seconds.
- **Memory** (7): The last reading will automatically be shown on the display for 2 seconds.
- **Ready for measurement** (8): When the unit is ready for measurement, the «°C» or «°F» icon will keep flashing.
- **Measurement complete** (9): The reading will be shown on the display (3) with the «°C» or «°F» icon; the unit is ready for the next measurement, when the «°C» or «°F» icon is flashing again.
- **Low battery indicator** (10): When the unit is turned on, the battery icon will keep flashing to remind the user to replace the battery.

## 5. Directions for Use

---

1. Press the ON/OFF button (4). The display (3) is activated to show all segments for 2 seconds.
2. The last measurement reading will automatically be shown on the display for 2 seconds with the «M» icon (7).
3. When the «°C» or «°F» icon is flashing, a beep sound is heard and the thermometer is ready for measuring (8).

- Place the measuring sensor gently at the center of the forehead. Please remove any hair, sweat or dirt from the forehead before measuring to improve the accuracy of the readings.
- Press the START button ② once and release it. A long beep sound will emit after 3 seconds to acknowledge the completion of measurement.
- Read the recorded temperature from the LCD display.

 **NOTE:**

- **Patients and thermometer should stay in similar room condition for at least 30 minutes.**
- Don't take a measurement while or immediately after nursing a baby.
- Don't use the forehead thermometer in high humidity environments.
- Patients should not drink, eat or exercise before/while taking the measurement.
- Before placing the thermometer probe onto the measurement area, remove dirt, hair, or sweat.
- Don't move the measurement device from the measuring area before hearing the termination beep.
- Use an alcohol swab to carefully clean the probe and wait for 15 minutes before taking a measurement on another patient.

- 10 short beeps and a red LCD backlight alert the patient that he/she may have a temperature equal to or higher than 37.5 °C.
- Always take the temperature in the same location, since temperature readings may vary according to locations.
- In the following situations it is recommended that three temperatures are taken with the highest one taken as the reading:
  1. New born infants in the first 100 days.
  2. Children under three years of age with a compromised immune system and for whom the presence or absence of fever is critical.
  3. When the user is learning how to use the thermometer for the first time until he/she has familiarized himself/herself with the device and obtains consistent readings.
  4. If the measurement is surprisingly low.

## 6. Changing between Celsius and Fahrenheit

This thermometer can display temperature readings in either Fahrenheit or Celsius. To switch the display between °C and °F, simply turn OFF the unit, **press and hold** the START button ② for 5 seconds; after 5 seconds, the current measurement scale («°C» or «°F» icon) will flash on the display ⑪. Change the measurement scale between °C and

°F by pressing the START button ②. When the measurement scale has been chosen, wait for 5 seconds and the unit will automatically enter the «ready for measuring» mode.

## 7. How to recall 12 readings in Memory Mode

This thermometer can recall the last 12 readings.

- **Recall mode ⑫:** Press the START button ② to enter Recall mode when power is off. The memory icon «M» will flash.
- **Reading 1 - the last reading ⑬:** Press and release the START button ② to recall the last reading. Display 1 alone with memory icon.
- **Reading 12 - readings in succession:** Press and release the START button ② consecutively to recall the last 12 readings in succession.

Pressing and releasing the START button ② after the last 12 readings have been recalled will resume the above sequence from reading 1.

## 8. Error Messages

- **Measured temperature too high ⑭:** Displays «H» when measured temperature higher than 42.2 °C / 108.0 °F.
- **Measured temperature too low ⑮:** Displays «L» when measured temperature lower than 34.0 °C / 93.2 °F.

- **Ambient temperature too high** (16): Displays «H» in conjunction with the «▲» when ambient temperature is higher than 40.0 °C / 104.0 °F.
- **Ambient temperature too low** (17): Display «L» in conjunction with the «▼» when ambient temperature is lower than 16.0 °C / 60.8 °F.
- **Error function display** (18): The system has a malfunction.
- **Blank display** (19): Check if the battery has been loaded correctly. Also check polarity (<+> and <->) of the battery.
- **Flat battery indication** (20): If the steady battery icon is the only symbol shown on the display, the battery should be replaced immediately.

## 9. Cleaning and Disinfecting

Use an alcohol swab or cotton tissue moistened with alcohol (70% Isopropyl) to clean the thermometer casing and the measuring sensor. Ensure that no liquid enters the interior of the device. Never use abrasive cleaning agents, thinners or benzene for cleaning and never immerse the device in water or other cleaning liquids. Take care not to scratch the surface of the sensor lens and the display.

## 10. Battery Replacement

This device is supplied with one lithium battery, type CR2032. Replace with a new CR2032 battery when the flashing battery symbol appears on the display (20). Remove the battery cover (21) by sliding it in the direction shown. Insert the new battery with the + at the top.

Batteries and electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

## 11. Guarantee

This device is covered by a **2 year guarantee** from the date of purchase. The guarantee is valid only on presentation of the guarantee card completed by the dealer (see back) confirming date of purchase or the receipt.

- The guarantee covers the device. The battery and packaging are not included.
- Opening or altering the device invalidates the guarantee.
- The guarantee does not cover damage caused by improper handling, a discharged battery, accidents or non-compliance with the operating instructions.

Please contact Microlife-service.

## 12. Technical Specifications

<b>Type:</b>	Forehead Thermometer FR 100
<b>Measurement range:</b>	34.0 - 42.2 °C / 93.2 - 108.0 °F
<b>Resolution:</b>	0.1 °C / °F
<b>Measurement accuracy:</b>	Laboratory: ±0.2 °C, 36.0 ~ 39.0 °C ±0.4 °F, 96.8 ~ 102.2 °F
<b>Display:</b>	Liquid Crystal Display, 4 digits plus special icons
<b>Acoustic:</b>	The unit is turned ON and ready for the measurement: 1 short beep. Complete the measurement: 1 long beep (1 sec.) if the reading is less than 37.5 °C / 99.5 °F, 10 short «beep» sounds, if the reading is equal to or greater than 37.5 °C / 99.5 °F. System error or malfunction: 3 short «bi» sounds.
<b>Memory:</b>	Auto-Display the last measured temperature 12 readings recall in the Memory Mode

**Backlight:** The display light will be GREEN for 4 seconds, when the unit is turned ON. The display light will be GREEN for 5 seconds, when a measurement is completed with a reading less than 37.5 °C / 99.5 °F. The display light will be RED for 5 seconds, when a measurement is completed with a reading equal to or higher than 37.5 °C / 99.5 °F.

**Operating conditions:** 16 - 40 °C / 60.8 - 104 °F  
15-95 % relative maximum humidity

**Storage conditions:** -20 - +50 °C / -4 - +122 °F  
15-95 % relative maximum humidity

**Automatic Switch-off:** Approx. 1 minute after last measurement has been taken.

**Battery:** 1 x CR2032 battery 3V

**Battery life-time:** approx. 2000 measurements (using a new battery)

**Dimensions:** 88.5 x 37 x 20 mm

**Weight:** 36 g (with battery), 33 g (w/o battery)

**IP Class:** IP21

**Reference to standards:** ASTM E1965;  
IEC 60601-1; IEC 60601-1-2 (EMC);  
IEC 60601-1-11

**Expected service life:** 5 years or 12000 measurements

This device complies with the requirements of the Medical Device Directive 93/42/EEC.

Technical alterations reserved.

According to the Medical Product User Act a biennial technical inspection is recommended for professional users.

Please observe the applicable disposal regulations.

### **13. [www.microlife.com](http://www.microlife.com)**

---

Detailed user information about our thermometers and blood pressure monitors as well as services can be found at [www.microlife.com](http://www.microlife.com).

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